


Price inquiry: +48 605999769, kontakt@openmedis.pl

Product code: SM02658

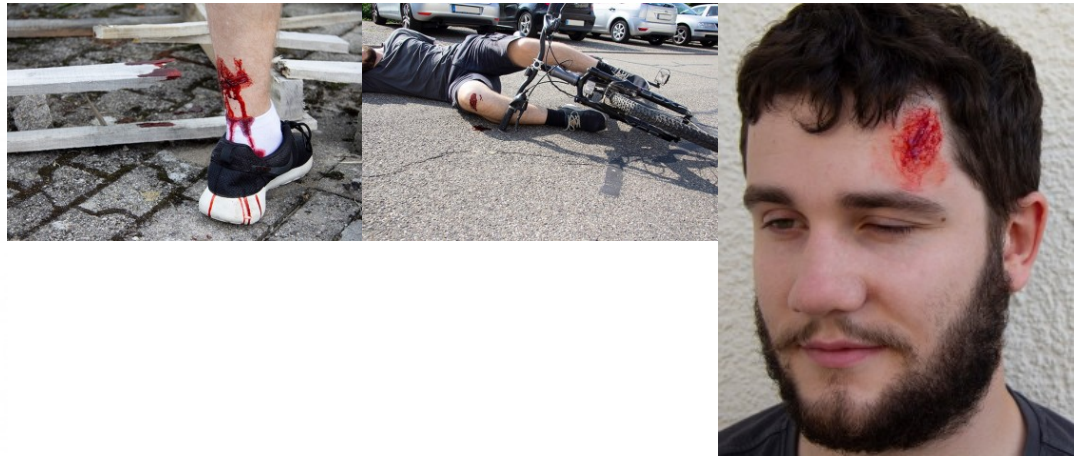

With this wound simulation kit, you can design incredibly realistic First Aid training wounds in minutes without knowing makeup techniques. The molds included in the set can be filled with any wound simulation paste. After the paste has dried, the filled molds are simply pressed into the desired position, the wounds adhere independently and without glue to the skin or to the training dummies. Just apply the included blood to the sealed wound and the wound is ready. If you want it to be even more realistic, you can achieve an even more real effect by applying "meat" paste to the wound. Stuck wounds can be treated several times, just remove the dressing, wash the wound and apply fresh blood. After training, the wound can be easily removed and disposed of. Consumables such as wound simulation paste, fixative powder, "meat" paste, and blood are sold separately. Includes cuts, abrasions and lacerations. Included materials include wound simulation paste, fixative powder, "meat" paste, artificial blood, isopropanol, paste spatula, and blood swabs. The perfect set for school medical services or youth emergency departments or fire brigades.